

# Chamber Brite™ - Powdered Autoclave Cleaner

**Tuttnauer**™  
Your Sterilization & Infection Control Partners

# Chamber Brite



## No Mess, No Hassles, No Measuring...

Chamber Brite powder is simply the easiest way to keep autoclaves clean and free of water deposit build-up. Designed specifically for steam sterilizers, Chamber Brite effectively cleans and descales the reservoir, chamber and tubing in minutes.

- Convenient, pre-measured packets
- Cleans reservoir, chamber and tubing
- Uses less storage space
- Replaces messy liquid cleaners



# Chamber Brite™ - Powdered Autoclave Cleaner

## Few simple steps to a hassle-free clean



### Cleaning Instructions:\*

1. Sprinkle Chamber Brite powder along the bottom of the COLD autoclave chamber.



2. Start a sterilization cycle WITH WATER and NO DRYING CYCLE.



3. Drain solution from reservoir and discard.

4. Fill reservoir and repeat steps 2 and 3 without Chamber Brite.



5. Wipe inside of chamber with damp cloth or sponge.



6. Fill reservoir with distilled water and sterilizer is ready for use.



For best results, clean autoclave every 20 cycles or once per week as needed. It is essential to follow the equipment manufacturer's directions for cleaning and maintenance.

\*For full instructions on how to use Chamber-Brite -follow the directions on the box.



Fully Automatic  
EZ Series



Manual  
M Series



Valueclave1730

### Tuttnauer Tabletop Autoclaves

Tuttnauer USA Co., Ltd., 25 Power Drive, Hauppauge, NY 11788  
Tel: (800) 624 5836, (631) 737 4850, Fax: (631) 737 0720  
Email: info@tuttnauerUSA.com, www.tuttnauerUSA.com

**Tuttnauer**<sup>™</sup>  
Your Sterilization & Infection Control Partners



LTR330-0008/CBF/USA/05.07